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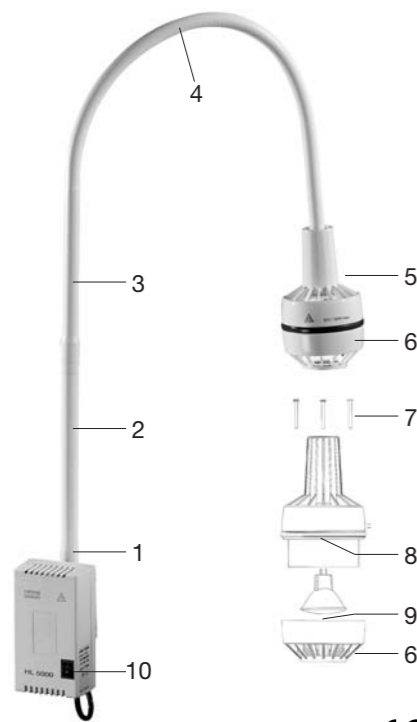
CE 2004/6/EWG

HEINE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)8152/38-0 · Fax +49(0)8152/38-202
E-mail: info@heine.com · www.heine.com
tec 8206 1/1.07

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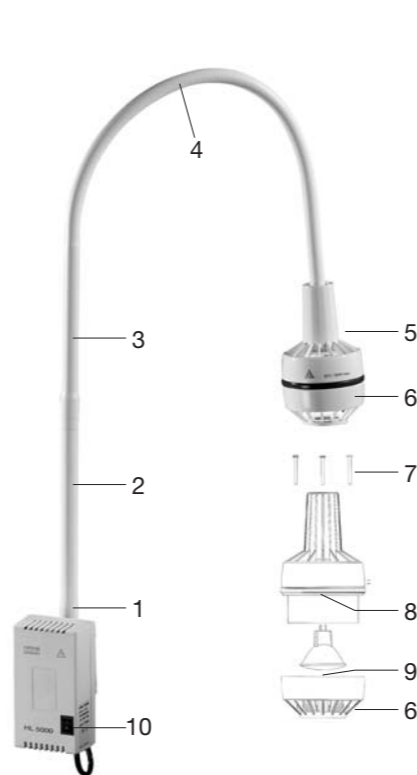
CE 2004/6/EWG

Warranty

We guarantee this device (except bulbs) for two years from date of purchase. We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product, the buyer must prove that any such defect was present at the time the product was delivered. This guarantee excludes any defects which occur because of wear and tear, incorrect use, the use of non-original HEINE parts, (especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance). Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

Bought from

date

Instructions

HEINE HL 5000® Workstation Light

Before using the HEINE HL 5000® read these instructions carefully and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

Safety information

Regulations for use: The HL 5000 is suitable for general illumination and can be left switched on for extended periods.

Safety: Do not open the transformer housing. Apart from a bulb change, all repairs must be carried out by qualified personnel. The HL 5000 should only be used when completely assembled.

Assembly: The HL 5000 can be mounted in any position. Ensure that the clamp is firmly attached. Do not obstruct the cooling slits in the transformer housing.

Safety in use: The mains voltage shown on the nameplate must correspond to the actual mains voltage. The HL 5000 may only be used with the protective cap (6) fitted to the illumination head. The illumination head must not be covered by foil or cloth etc. The light should not be used in environment where an explosion hazard is present. Do not shine the light directly into the eyes. Position the illumination head at least 20 cm from the object. Do not leave the light switched on without supervision.

Assembly/Installation

Installation instructions and materials are included in the packaging.

- Wall-mounting (instructions S-249.07.01)
- Clamp-fitting (instructions S-249.23.01)
- Wheeled stand (instructions S-249.33.01)

Plug in the mains cord only after assembly is completed.

Instructions for use

The swivel arm of the HL 5000 has two flexible (2, 4) and one rigid section (3). The flexible sections can be bent in any direction at up to 90° (2) or 180° (4). They should not be bent to a greater degree to avoid wear. To position the light, grasp the flexible section behind the illumination head (5). Switch on by means of switch (10).

Bulb change

- Disconnect the mains cord plug.
- Allow the bulb to cool.
- Unscrew the three screws behind light head (7) and remove the cap (6).
- With thumb and forefinger grasp the bulb and pull it out of the socket.
- When inserting the new bulb, ensure that the contact pins are not bent. The contact pins must be pushed into the socket as far as they will go. Do not touch the safety cap.
- Replace the cap (6) and ensure that the projection (8) engages in the groove (9).
- Tighten the 3 screws to refit the cap (6).

Important note! The safe and reliable operation of the light can only be guaranteed if an original HEINE bulb is fitted.

Changing a fuse

Disconnect the mains cord plug! The fuses are located in the fuse compartment of the integral plug. Use only the fuses listed in the technical specification.

Cleaning

Disconnect the mains cord plug! Clean only when the unit is cool. To clean the HL 5000 use only a soft cloth with soapy water or other non-aggressive solution.

Technical specification

Input: (see nameplate)	Voltage	Fuse
	100 V~ 50/60 Hz	2 x 1,25 A slow
	120 V~ 50/60 Hz	2 x 1,25 A slow
	127 V~ 50/60 Hz	2 x 1,25 A slow
	230 V~ 50/60 Hz	2 x 630 mA slow
Power consumption:	70 VA	
	12 V / 50 W	
Output:	12 V / 50 W	
Bulb:	12 V / 50 W	
Light beam angle:	10°	
Bulb life:	approx. 4000 hours	
Colour Temperature:	approx. 3100 K	
Conditions of use:	continuous (ambient temperature up to +40 °C)	
Classification:	Type B ↑	
Weight with wall mount:	2,6 kg	
Weight of clamp:	0,2 kg	
Weight of wheeled stand:	3,2 kg	
Length of Flexible arm:	114 cm	
International Protection:	IP 20	
Equipment:	Class I	

Accessories and Spare Parts

Non-Tip wheeled stand, metal base	T-05.27.106
Lightweight wheeled stand L, polycarbonate base	T-05.27.108
Wall mount	T-05.27.102
Clamp mount	T-05.27.104
Spare bulb 12V/50 W halogen	T-05.27.111

Applicable Standards: EN 60 601-1 (1990)
EN 60601-2-41 (2000)